RESCUE Life 9

Think different, be different

defibrillator

Technology for diagnosis and emergency

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Design

Born from expert Italian designers, its strength is simplicity and beauty. Attractive line, ergonomic form, studied in every detail, Rescue Life 9 is the first professional defibrillator totally MADE IN ITALY that respects the traditions of elegance and comfort. Thanks to the lightweight and transport facility this defibrillator is the perfect device in the hard challenges. Quality and safety is the mix of our expertise, applied to Rescue Life 9 as well to all our products line. The IPX4 standard permits the operation under adverse climatic situations.

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Technology

Rescue Life 9 is a new top product evolved from the improvements to the defibrillators manufacture. From more than 25 years' experience, Rescue Life 9 is the advanced solution for emergency medicine. Ergonomic form, extremely robust, it is the first professional defibrillator totally Made in Italy. Auto-test mode allows to verify the proper functions of the device, battery status and charge-discharge system with the possibility of real time data management and report printing.

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Usability

Close to the needs of the modern emergency medicine, Rescue Life 9 guarantees impact-proof 8.4" display, powerful battery up to 5 hours continuous monitoring and different product configurations. Individuals can choose from basic to fully equipped models among the available optional: up to 12 leads ECG monitoring, AED & Advisory working mode, external pacemaker, pulse oximeter, non-invasive blood pressure, entidal CO2 and temperature.

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A different point of view

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Display

Brilliant, high-resolution TFT LCD color 8.4" 800x600 dots display allow the simultaneous visualization in real time of 3, 6 or 12 leads ECG and plethysmograph waveform with SpO2 sensor connected. ECG leads error message when cable is disconnected. Customizable ECG display traces in setup menu. Three operating modes: manual, AED, Advisory, with capnography waveform.

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Fast connections

Users can choose between ergonomic reusable paddles, suitable for the manual defibrillation in both adult and pediatric patients or disposable pads for AED defibrillation. Both types are equipped with a single fast lock connection for reliable rescue operations.

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Intelligent operator support

Visual messages area on screen, different colors for every specific function and clear voice prompts support the operator during the most difficult situations. Simple menu with dedicated functions keys help the use with direct input, the double battery pack assures long working time operation.

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The combined defibrillator monitor Rescue Life 9 is designed to be used by out-of hospital and hospital users. Having an innovative and fully featured design, it provides for defibrillation and monitoring in a single portable device.

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High performance ecg analysis

Rescue Life 9 can filter the ECG signal automatically thanks to two modes: **DIAGNOSTIC**: ECG signal processing recommended for interpretation and diagnosis of ECG traces by the qualified personnel. **MONITORING**: digital filtered ECG signal, rejects the most parts of noises and artefacts caused by patient motions or by other physiological process of the body.

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Memory and data management

Simple commands, extremely intuitive and easy to use. Immediate access to main setup parameters using only the trim knob and function keys. The internal memory is able to record ECG data and events. Printing can be done in manual or in automatic mode. In the automatic mode the printing initiates when the energy charging starts (recording the ECG signal before the shock) and will continue 6 seconds after the shock was delivered.

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Alarm mode

Configurable alarms for every single function, if the set limits are exceeded, an alarm message will appear in the dedicated area on the display. The automatic printing function allows to have a report before and after the shock release and automatically of all 12 leads ECG.

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AED & Advisory mode

Simple step 1-2-3 supports the operator during use in AED mode. Voice prompts and visual messages guide the procedure of CPR and release the shock. Advisory mode automatically and continuously analyzes the ECG rhythm with ECG cable or pads connected to patient and finally it advises if the shock is necessary.

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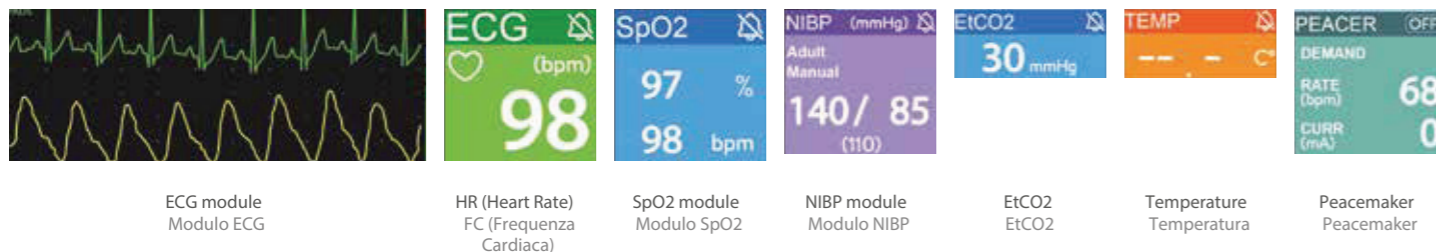
A different point of view

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The combined monitor defibrillator Rescue Life 9 is designed to provide complete monitoring of main patient's parameters like 12 channels ECG, HR, SpO2, NIBP, Temperature and EtCO2. Rescue Life 9 can also perform synchronized cardioversion and provide external pacemaker in fixed or demand mode.

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Optionals

Optimized and comfortable carrying bag, SpO2 module, multifunction disposable defibrillator pads for adult and pediatric, 10 leads ECG cable, pacemaker module, NIBP module, EtCO2 module, mounting bracket 10G tested.

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Technical specifications

GENERAL REQUIREMENTS

Dimension & weight • 335 x 225 x 360 mm, • 6,5 Kg with dual batteries
Electrical • AC 100-240 VAC, 50/60 Hz • Rechargeable dual batteries: 15.0 V 3.2 Ah Lithium-ion
Battery Operation Time • >5 hours, >10 hours with dual batteries
Battery Charge Time • Max. 3 hours
Capacity • 150 shocks with single battery or 300 shocks with dual battery at 230J fully charged
Software • Voice and texts multi language
Memory • Internal flash for ECG data, events and usage review

ENVIRONMENTAL

Temperature • Operational -5 ~ 46C, • Storage -20 ~ 50C, • Relative humidity 10 ~ 95%
Isolation • ECG type CF, Defibrillation type BF
Water proof • Class IP54

DEFIBRILLATION

Waveform • Biphasic Truncated Exponential (BTE), HiCAP technology (Large storage capacitor)
Impedance • Compensated, range 25 - 200 Ohm
Charge time • 6 sec for 230 J with battery fully charged

AED MODE

Energy • Fixed energy at 200 J
Protocol • ERC 2015 CPR Guidelines with voice and text prompts
Shockable rhythms • VF with amplitude > 0.15 mV and VT with rhythm >150 bpm
Sensitivity • Shockable rhythm-VF > 95%, Shockable rhythm-VT > 75%
Specificity • According to ERC 2015, Normal sinusoidal rhythm > 99% • Asystole and other non-shockable rhythms > 99%

MANUAL MODE

Energy range • 1 - 230 J (from 1-10 J in 1 J steps; from 10 to 230 J in 10 J steps)
Commands • Multifunction trim knob. Charge and shock button directly in the front panel for hands free defibrillation
Paddles • Reusable adult & pediatric defibrillation pads with charge/shock command • Disposable defibrillation pads and internal defibrillation pads (optional)
Operating mode • ECG « R » wave synchro or async mode
Indicators • Battery and main led indicators • Clear and visible backlight color buttons

DISPLAY

Type • High contrast TFT color LCD 8" (800X600dots)
Information • Operational information, ECG traces, Spo2 value, NIBP values, etCO2, TEMP, Heart Rate indicator, filters, alarms and batteries status
Display mode • Can select 3, 6, 12 ECG leads

ECG MONITORING

Bandwidth • 0.05 - 120 Hz (-3 db) in diagnostic mode • 0.6 - 40 Hz (-3 db) in monitoring mode
CMRR • > 98 db
Input impedance • > 50 Mohm
Input • Patient cable 3 leads (selectable I, II or III) - optional
• Patient cable 5 leads (I, II, III, aVL, aVF, aVR, V) - standard
• Patient cable 10 leads (I, II, III, aVL, aVF, aVR, V1.....V6) - optional
Gain • 2,5, 5, 10, 20, 40 mm/mV
Speed • 5, 10, 25, 50 mm/sec
Filters • 50/60 Hz, EMG Filter, base line
HR range • 20 - 300 bpm (~ 2%), digital readout on the display
Alarm • HR max settable 250 bpm; HR min 20 bpm

SPO2 (OPTIONAL)

SpO2 range • 0 - 100%
HR range • 30 - 250 bpm
Accuracy • 70 - 100% ~ 2% for adults with finger clip sensor
Alarm • Adjustable min 50%

NIBP (OPTIONAL)

Technique • Oscillometric
NIBP Accuracy • Meets ANSI/AAMI SP10-2002, EN 1060-4
Patient Application • Adult/Paediatric/Neonatal
Systolic Range • Adult: 40-260mmHG, Paediatric: 40-160 mmHG, Neonatal • 40-130 mmHG
Range MAP • Adult: 26-220 mmHG, Paediatric: 26-133 mmHG, Neonatal • 26-110 mmHG
Diastolic Range • Adult: 20-200 mmHG, Paediatric: 20-120mmHG, Neonatal • 20-100 mmHG
PR RANGE • 30-220 BPM
PR Accuracy • +/-2% or +/-3 bpm
Operating Modes • Manual, Long Term Automatic, Stat, Service
Auto Interval Periods • 1,2,3,4,5,10,15,30,60 and 90 minutes

PRINTER

Type • Integrated thermal 3 channels printer for ECG traces and events documentation hardcopy including HR/SpO2/NIBP/etCO2/TEMP values
Paper speed • 5, 10, 25, 50 mm/sec
Paper width • 58 mm
Operating model • Manual, automatic (12" pre and post shock recording)

EXTERNAL PACEMAKER (OPTIONAL)

Type • Rectangular wave
Operating mode • Fixed on demand
Pulse rate • 30 ppm to 250 ppm, adjustable in steps of 5 ppm
Impulse duration • 22.5 ms
Pulse current • 0 to 180 mA adjustable in steps of 5mA
Amplitude • Max 200 V

ENTIDAL CO2 (OPTIONAL)

Mainstream technology for both intubated and non-intubated patients
Measurement range • 0 to 150mmHg, accuracy ±2%
Respiratory rate • 0 to 150 bpm

TEMPERATURE (OPTIONAL)

Probe type • YSI400 compatible
Range • 0 to 50 C
Resolution • ±0.1 C

Standard • IEC 60601-2-4; IEC 60601-1; IEC 60601-1-2
CE Mark EEC 93/42 • Medical device, Class IIb

SPECIFICATIONS AND CONTENTS OF THIS DOCUMENT SUBJECT TO CHANGE WITHOUT NOTICE

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