STARSONIC FLOW

Art. 126

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ULTRASONIC WASHING MACHINE FOR HOLLOW INSTRUMENTS

LIARRE introduces a new concept in the cleaning of rigid accessories for endoscopes, aspiration tubing, reamers, accessories for urologic and gynaecologic exams, etc.

It's a fully digital ultrasonic bath with a capacity of 40 lt. It is different to the standard Starsonic range from Liarre, in that it is a "forced pump flow ultrasonic bath" utilising a powerful centrifugal pump. It is supplied with a large stainless steel basket with rubber outlets, allowing you to connect tubings of any diameter.

The organic residus can be easily eliminated from the filter, which can be reached from outside.

This new bath is a major step forward from manual cleaning and the benefits are seen very quickly:

1. An end to the drudgery of manual cleaning of rigid endoscopes and other hollow instruments;
2. Once connected the cleaning process is fully automatic. As the operator is only needed at the beginning and end of each process, they are free to complete other processes in between, thus reducing operating costs;
3. Everyone is very aware of infection control and our automatic machine keeps operator contact with contaminated instruments to a minimum;
4. Manual cleaning creates aerosols that can contaminate both the operator and the room around them; our machine operates in a closed environment thus keeping contamination to a minimum;
5. It is very difficult to guarantee meticulous cleaning in any manual procedure. Our Starsonic Flow, with its automatic process can guarantee a fully cleaned instrument every time;
6. In a very busy endoscopy unit more than one person may be needed to clean endoscopes. With more than one Starsonic Flow in the cleaning room, one person can clean many endoscopes at the same time, thus allowing vital staff resources to be freed up for work elsewhere;
7. Because of the flexible rubber outlets fitted in the basket, you only need one machine to fit all rigid endoscopes, rather than one machine for each different type of rigid endoscope used in a department, thus keeping equipment costs to a minimum.

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